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## SYSTEM INSTALLATION MANUAL

### SYSTEM REQUIREMENTS

- Operating system: Windows XP, Windows Vista, Windows 7, Windows 8
- Processor: Intel® Pentium® IV 3200 MHz or higher
- Browser: Microsoft® Internet Explorer® 5.0 or higher
- Space available on Hard disk: 160 Mbyte or higher
- RAM Memory: 1024 Mbyte or higher
- Video resolution: 800x600 or higher and 65000 colours or higher
- CD ROM or DVD ROM devices
- Built-in or external WiFi network card
- Built-in or external SD card reader

### PROCEDURE FOR INSTALLATION


1. Switch off the autoclave
2. Remove the original SD card from the New E9 Recorder and connect it to the PC
3. Open the E-Memory Software and wait for all the data on the SD Card to be downloaded.  
To check that all the data have been downloaded, simply click on the icon on the welcome page and check the communication icon: if the inactivity message appears for more than a few seconds, this means that the download of data has been completed. The download process can take from a few minutes to 1-2 hours, depending on the PC used and the amount of data being transferred
4. Closing the E-Memory Software
5. Create a folder on the desktop and rename it "SD card data"
6. Open the SD card folder and copy:
  - the BACKUP folder
  - the CYCLES folder
  - all the \*.OVO files
  - the MAINTEN.EUR file
  - the INSTALL.EUR file
  - the EMPTY.EUR file
  - the OPER\_PC file if present
7. Paste the files in the "SD card data" on the desktop. Remove the original SD card and connect the E-WiFi SD card to the PC

8. Remove the original SD card and connect the E-WiFi SD card to the PC
9. Access the E-WiFi SD card and paste in the root section (i.e. NOT inside any folders but directly on the SD) the files copied from the original SD card
10. Remove the E-WiFi SD card from the PC and insert it in the New E9 Recorder
11. Switch on the autoclave
12. Open the connection centre and manually connect to the network which will be called "EWF\_....." and enter the password: 12345678
13. Open the web browser and enter the address below: 192.168.0.1
14. Check that the E-WiFi SD card page appears with the saved cycles, then close the browser
15. Insert the installation CD in the PC and install the E9-Link software, following the setup wizard
16. Check that the E-Memory Software is closed
17. Open the E-WiFi interface and click "Save cycles now!". The icon on the application bar will turn yellow for about a minute and then turn white.  
When downloading is complete, the icon on the application bar will turn green. The download time can vary from a few minutes to various tens of minutes, according to the amount of data, the distance between the PC and the sterilising unit, and the type of PC and WiFi card
18. When downloading is complete, re-open the E-Memory Software


## THE SYSTEM TRAY ICONS

	Connection to the autoclave in progress
	Download of data in progress
	Download completed correctly, cycle in progress
	Download completed correctly, no cycle in progress
	Connection error


## USER INTERFACE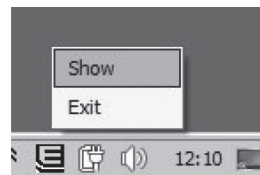


- Right click the icon:
- **Show** opens the interface
  - **Exit** closes the programme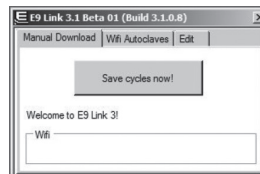

**Save Cycles now!** download the cycles immediately from the connected sterilising unit, without waiting for the programme moment (after timeout)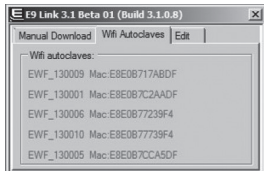

List of autoclaves  
WiFi maximum 5 New E9 Recorder autoclaves  
\*(card not detected on Windows XP)

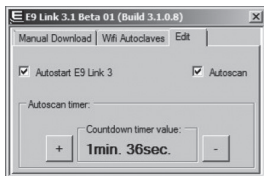- **Autostart** if flagged, 9-Link will run when the PC is booted up (default setting)  
- **Autoscan** downloads the cycles automatically from all the autoclaves connected via E-WiFi (default setting)  
- **+** increases and decreases the timeout value (time interval after which the programme searches for new sterilisation cycles to be sent to the PC via WiFi); minimum 4 minutes, default settings

## TROUBLESHOOTING

PROBLEM	SOLUTION
the SD card E9-Link page will not open on the browser	<ul style="list-style-type: none"> <li>- check that the autoclave is switched on</li> <li>- check that the WiFi network card is enabled</li> <li>- check that the correct address has been entered</li> <li>- check that you are connected to the SD card E9-Link network</li> </ul>
the system bar icon turns red	<ul style="list-style-type: none"> <li>- check that the autoclave is switched on</li> <li>- check that the WiFi network card is enabled</li> <li>- check that the distance between the autoclave and the PC is not too long</li> <li>- on Windows XP: check which WiFi network the PC is connected to</li> </ul>
the icon remains white for a long time	<ul style="list-style-type: none"> <li>- the download is slow: check the distance between the PC and autoclave and that no other devices are connected to the SD card E9-Link network</li> </ul>

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