DE | GB | FR | ES | NL | IT

Baclyser[®] neo T

User Manual

DE

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GB

Instruction for use

FR

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ES

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NL

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IT

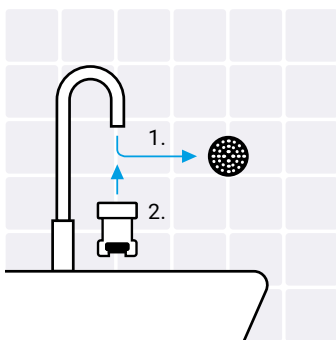
Istruzioni d'uso

Installation / Installation / Montage / Montaje / Montage / Montaggio

1. Installation Schnellverschlusskupplung / Installation quick release coupling / Installation du raccord rapide / Instalación del acoplamiento rápido / Installatie snelkop- peling / Installazione attacco rapido


Händedesinfektion
Hand disinfection
Désinfection des mains
Desinfección de manos
Desinfectie van de handen
Disinfezione delle mani

DE 1. Strahlformer entfernen. 2. Passende Schnellverschlusskupplung mit Dichtung installieren. / **GB** 1. Remove the aerator. 2. Install suitable quick-release coupling with seal. / **FR** 1. Enlever le brise-jet. 2. Installer le raccord rapide approprié avec le joint d'étanchéité. / **ES** 1. Retirar aireador. 2. Instalar un acoplamiento rápido adecuado con junta. / **NL** 1. Verwijder de straalregelaar. 2. Monteer een geschikte snelkoppeling met afdichting. / **IT** 1. Rimuovere l'aeratore. 2. Installare un adatto giunto a sgancio rapido con guarnizione.

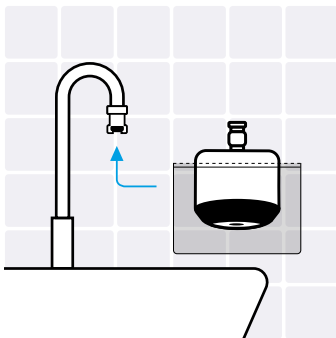2. Filterinstallation / Filter installation / Installation du filtre / Instalación del filtro / Filter installatie / Installazione del filtro

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Product description

The Baclyser® neo T is a CE-marked, sterile¹, point-of-use tap filter for use in high-risk areas of medical facilities for the prevention of bacterial nosocomial infections caused by *Pseudomonas aeruginosa*, *Legionella pneumophila*, *Stenotrophomonas maltophilia* or *Staphylococcus aureus* in immunocompromised and immunosuppressed patients. The effect is achieved by interrupting the transmission path of these germs through filtration of the water and protection against retrograde contamination when using the water taps through hygienic design and antimicrobial protection functions. The Baclyser® neo T is available in three different service lives (1M, 2M and 3M) and in two different outlet variants each (laminar and rose).

Technical data

Size: 49.3 mm x Ø 52 mm

Retention: 7 log steps ²

Membrane: Flat sheet membrane,
pore size 0.2 µm

Operating time: up to 31, 62 or 93 days
(1M, 2M or 3M) ³

Flow rate: 15.5 l/min at 5 bar,
13.1 l/min at 3 bar

Max. operating pressure: 5 bar ⁴

Max. operating temperature: 60 °C
(at 75 °C ≤ 90 min over operating time)

Connection: Quick-release coupling

Chlorine resistance: ≤ 10 ppm ⁵

Chlorine dioxide resistance: ≤ 0,8 mg/l

¹ Sterile filtration: Pathogen retention ≥ 10⁷ *B. diminuta* CFU/cm² filter membrane surface (= 7 log steps).

² Measured in accordance with the international ASTM F838-15a standard – Standard Test Method for Determining Bacterial Retention of Membrane Filters Utilized for Liquid Filtration.

³ Depending on water quality. If you use the Aqua free filter change service, the filter will be changed every 4 or 8 weeks.

⁴ The Baclyser® neo T must not be connected to low pressure units (boiler).

⁵ Continuous dosage of ≤ 10 ppm over operating time; short-term (1h) high dosage (400,000 ppm) for chemical disinfection.

Adapter and functional materials

Quick-release coupling M24,
external thread (incl. seal),
with AquaStop,
(Art. No. 100298)

Quick-release coupling M22,
internal thread (incl. seal),
with AquaStop,
(Art. Nr. 100296)

Quick-release coupling M24
short, external thread (incl. seal),
without AquaStop,
(Art. Nr. 102668)

Further adapters and functional materials on request

Maintenance and cleaning

The Baclyser® *neo* T requires no maintenance within the specified operating time. The outside of the filter can be cleaned using common surface disinfectants. To prevent retrograde contamination, daily wipe disinfection of the Baclyser® *neo* T is recommended.

When replacing the Baclyser® *neo* T, a small amount of unfiltered water will flow out. Over time, this will lead to deposits. For this reason, adapters and connections should be checked at regular intervals for functionality and proper condition and replaced as required.

In the case of a deviation from the specifications or a damage of the package, the product should be returned to Aqua *free*, along with the “Defect Report” form (download at aqua-free.com).

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Aqua free Solutions Ltd.
Aqua free S.A.R.L.
Aqua free B.V.