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## Instructions for use – operating theatre step

Single operating theatre step    Double operating theatre step

Equipped with non-slip, antistatic feet.

### Intended use

The operating theatre step serves as a platform for standing at tables or shelves. The maximum load of the operating theatre step is 150 kg.

Damage caused by improper use results in loss of liability and of guarantee claims.

### Safety instructions

The operating theatre step may only be used on dry, even floors. Always wear non-slip shoes when standing on the operating theatre step.

There is a danger of slipping on damp and wet surfaces. Immediately clean and dry any damp and wet surfaces.

Steps standing on busy passageways are a tripping hazard. Store the steps safely when not in use.

The operating theatre step may only be used in a technically sound condition, with risks and safety in mind, for its intended purpose and according to the operating instructions.

### Rating plate

The rating plate is located in front of the first step, on the right.

- |   |                          |   |                       |
|---|--------------------------|---|-----------------------|
| 1 | Manufacturer and address | 4 | Weight                |
| 2 | Type designation         | 5 | Load bearing capacity |
| 3 | Order number             |   |                       |

### Cleaning

#### **i** ATTENTION

##### **Stainless steel: unsuitable cleaning agents**

The stainless steel surfaces of the device may be damaged by incorrect handling and rust may form. Therefore, take the following precautions when using the device:

- ▶ Never use the following for cleaning: chlorinated detergents, scouring powder or other dry cleaners, cleaning wool, steel wool pads or sharp-edged objects.
- ▶ Read the special care instructions for stainless steel and follow the instructions contained in them. You can download the care instructions at [www.hupfer.com/downloads](http://www.hupfer.com/downloads).

#### **👉** NOTE

##### **Suitable cleaning agents**

The manufacturer cannot assess the suitability of specific detergents or disinfectants.

- ▶ Check the notes on the detergents or disinfectants to find out whether they are suitable for the materials in the product.
  - ▶ If in doubt, ask the manufacturer of the cleaning agent if it is suitable for the materials.
- 
- ▶ Clean and disinfect the surfaces of the device after use or as required.
  - ▶ For wet cleaning, use a soft cloth or an uncoated sponge.
  - ▶ Use degreasing liquid cleaners.
  - ▶ Disinfect with commercially available disinfectants.

Check the screw connections annually. Tighten the screw connections if necessary.

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